Total Blood Bank Processing Solutions

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The blood bank centrifuge DP-2065-R PLUS model is a refrigerated centrifuge designed for big and small blood processing centers.

The microprocessed refrigerated centrifuge DP 2065 R Plus with capacity for six glasses, 1200 ml each, to use 6 bags (double, triple and quadruple bags and for Top & Bottom and Buffy Coat system).

It is ideal to perform component separation procedures such as red cells platelet rich plasma, platelet concentration or cryoprecipitation.

It can also be used to perform lots of special blood bank procedures, like Buffy Coat separation.

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DP-2065 R Plus

- Capacity 6 buckets for 1000 ml each.
- Heavy-duty brush-less induction motor.
- Safety lock lid
- Safety Alarms: imbalance, power failure, over speed and lid open with safety interlock.
- Continuous display of max RCF.

Options

- Opty-System adaptors
- Plastic buckets adaptors.
- Capacity for bottles of 1 liter
- Wide range of adaptors for tubes from 500 ml. to 0,5 ml.

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The new panel MEMOSOFT V added the reliability and security recognized as a communication system with the user through graphical displays for the equipment handling. The new system allow store all processed units download data to a PC processes performed in order to have maximum traceability in the processing of blood units. With this new process handling system will be able to know when it each blood unit was processed can also store data process unit centrifuged with date, time, method and duration of centrifugation processes to be traceable and safe.

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Predefined processing programs - Integration function.
Storage up to 50 programs including the following variables:
PROGRAM NAME
SPEED: programmable depend on the rotor model.
TEMPERATURE: programmable from -10° C to 40° C.
TIME: programmable from 1 to 99 minutes & Hold.
ACCELERATION: programmable 100 levels.
BRAKING: programmable 100 levels
PC CONNECTION: store data with exclusive software.

Variable Indicators
SPEED: four digits indicator.
TIME REMAINING: two digits indicator.
TEMPERATURE: tree digits indicator.

-Continuous RCF indication -Non volatile memory
 -Lock lid security -Imbalance disconnection and indication
 -Brush-less induction motor -Free CFC refrigeration system.
 -Security system with optical and audible alarms for imbalance and incorrect closing lid which stops the centrifugation automatically.

OPTIONS AND ACCESSORIES
 -Bar code reader
 -Connection to PC cable
 -Data processing software in PC
 -Plastic buckets